

Instruction for use

Manifolds



Description of product

- 3- or 5gang manifold with holding plate for easy and secure attachment to the IV pole with a special clamp. They are transparent, lipid resistant and pressure resistant till 1.5 bar. Handles are with different colors to differentiate the monitoring lines.
- REF: 701 xxx

General notes:

- Read carefully instructions before using.
- The product may be shown commitment only according to the described dedication. The application of the product only may be carried out of a qualified doctor or under his instruction.
- Store in a cool and dry place with ambient temperature.
- Do not expose packed products to direct sunlight.
- Do not use damaged products.
- Do not use products from damaged packages.
- Do not resterilize.
- Do not use if expiry date on the packaging (label) has expired.
- The connections may have become loose during transport and storage or during sterilisation process. Please check all connections and ensure that all are tight.

Special notes:

Gravity use.

In spite of all care we took to manufacture and control our products, we advise you to check the packaging and device integrity before use.

We advise exceeding the maximum pressure may result in device failure.

All our stopcocks and manifolds have 6% Luer Lock connections in order to be easily connected to the others devices.

Return

Prior to the return of a product due to a quality complain, you must decontaminate it and add to your delivery the corresponding decontamination certificate.

In the absence of this certificate, your complain will be automatically rejected and the related product immediately destroyed.

Other notes:

Do not allow to the catheter to come into contact with acetone and disinfectant agents, as this can weaken the material or cause leaking or air aspiration.

Waste management:

After use the product becomes according to the regulations disposed for infectious waste or according to the national or regional determinations.

Excluding of liability:

We declare under our sole responsibility that the product meets the provisions of the Council Directive 93/42/EEC concerning medical devices which apply to them. Due to the biological differences of the patients the effectiveness cannot be guaranteed absolutely. Since we have no control over the use conditions, diagnosis and indication position as well as application and use of the product we can neither guarantee the success nor exclude the appearance of side effects.

Intra doesn't have to be held responsible for any loss, damage or injury of any manner, they may be made direct or indirect, or resultant damages which can arise in connection with the product or from its use.

Intra doesn't provide any replacement for costs which arise in connection with this result for the buyer or a third party with the use of the product. This exclusion also refers to doctor-, hospital-, for drug and additional costs or resultant damages. intra special catheters GmbH disclaims the responsibility for direct or indirect injuries that are a consequence of the product being modified or wrongly used, or of the disposable product being used more than once.

Should the product used several times (Contrary to our determinations), we don't ensure the biological, chemical, physical and functional specifications and qualities of the product.



intra special catheters GmbH

Oststr. 2 - D-66780 Rehlingen-Siersburg - Germany